

Instructions for Use












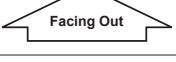
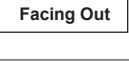

Solo Octo LiftSheet is a disposable lift sheet for lifting and transferring patients in a horizontal position. It is only intended for use with the Liko OctoStretch accessory, Prod. No. 3156056.

IMPORTANT!

Lifting and transferring a patient always involves a certain level of risk. Read the instructions for use for both the patient lift and the Liko OctoStretch before use. It is important to completely understand the contents of the instructions for use. The equipment should only be used by trained personnel. Exercise care and caution during use. As a caregiver, you are always responsible for the patient's safety. If something is unclear, contact the manufacturer or supplier.

Symbol Description

These symbols can be found in this document and/or on the product.

Symbol	Description
	Warning; this situation requires extra care and attention.
	Read instructions for use before use
	CE mark
	Legal manufacturer
	Date of manufacturing
	Product Identifier
	Serial Number
	Medical device
	Do not wash
	Latex Free
	Do not use Product (Only Solo products and visible after wash when use is prohibited).
	Direction upwards and symbol on outside of product
	Symbol on outside of product.
 <small>(01) 0100887761997127 (11) YYMMDD (21) 012345678910</small>	GS1 Data Matrix Barcode that may contain following information. (01) Global Trade Item Number (11) Production Date (21) Serial Number

Safety Instructions

 Medical Device Class I Product

Product Changes

Liko's products undergo continuous development, which is why we reserve the right to make product changes without prior notice. Contact your Hill-Rom representative for advice and information about product upgrades.

Design and Quality by Liko in Sweden

Liko is quality-certified in accordance with ISO 9001 and its equivalence for the medical device industry, ISO 13485. Liko is also certified in accordance with environmental standard ISO 14001.

Notice to Users and/or Patients in EU

Any serious incident that has occurred in relation to the device, should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

Care and Inspection

Check the product before each use. Check the following points with regard to wear and damage:

- Fabric
- Straps
- Seams
- Loops

⚠ Do not use damaged lifting accessories.

If anything is unclear, please contact the manufacturer or supplier.



DO NOT WASH. The caution label situated on the back of the lift sheet displays the symbol DO NOT WASH. If the lift sheet is washed, this label disintegrates to display the symbol DO NOT USE. The lift sheet must then be disposed of.



Expected Life Time

⚠ IMPORTANT!

Solo Octo Liftsheet is a disposable item that is intended for individual use only.

When the correct size has been determined for a particular patient, write the patients name on the product label on the back of the lift sheet.

The Solo Octo Liftsheet should no longer be used:

- if it is soiled or if it is suspected of being contaminated
- if it is damaged
- when the patient no longer requires it.


Recycling Instructions

The entire product should be sorted as Combustible waste.

Hill-Rom evaluates and provides guidance to its users on the safe handling and disposal of its devices to aid in the prevention of injury, including, but not limited to: cuts, punctures of the skin, abrasions, and any required cleaning and disinfection of the medical device after use and prior to its disposal. Customers should adhere to all federal, state, regional, and/or local laws and regulations as it pertains to the safe disposal of medical devices and accessories.

If in doubt, the user of the device shall first contact Hill-Rom Technical Support for guidance on safe disposal protocols.

Overview Solo Octo LiftSheet™

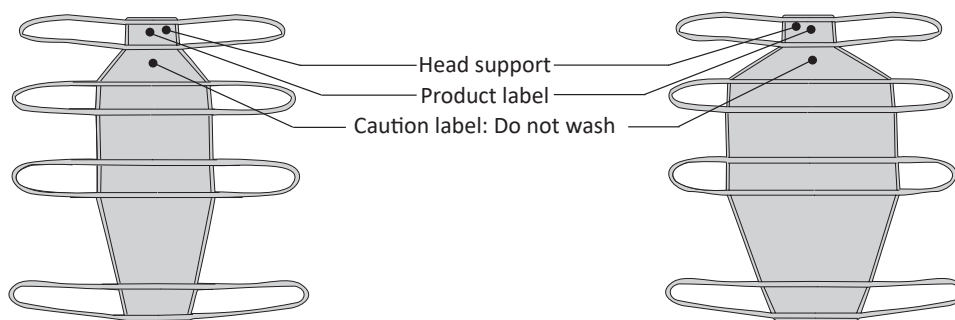

3683905 - Solo Octo Liftsheet, medium
Sheet size: 500x1600 mm / 19.7x63 inch. (WxL)

3683906 - Solo Octo Liftsheet, large
Sheet size: 900x1700 mm / 35.4x67 inch. (WxL)

Maximum load: 200 kg / 440 lbs.



www.hillrom.com

Liko AB
Nedre vägen 100
975 92 Luleå, Sweden
+46 (0)920 474700

Liko AB is a subsidiary of Hill-Rom Holdings, Inc.

Enhancing outcomes for
patients and their caregivers:

Hill-Rom